

Quick shot

Infrared Thermometer
Model: HT-F03A

A German Technology Product



Different back light color LCD

- Quick shot, Non Contact, Easy operation.
- Fast and accurate results.
- No Cleaning , No wiping.
- Reading Options in °C/°F.
- Fever alarm, Digital LCD Display.
- 32-Group of Memories.
- Diff. Modes of Operations.
- Auto Off - Power Saver.



Handle with care:

1. The protective lens of probe is the most damageable part of the infrared thermoscope. Hence it must be taken good care.
2. Cleaning method of probe lens: use the cotton bud or soft cloth with water or alcohol to wipe the lens lightly.
3. never charge the batteries or throw any of them into fire. Please discard the used batteries at a specified location for recycle. Using the batteries with a bad quality may cause fire or explosion.
4. When the product is not used for a long time, the batteries shall be removed.
5. The product may not be immersed into water or exposed to the sun directly.
6. Never hard throw or impact the product. Otherwise, it will be damaged .
7. If the measurement is made without an effective distance or an alignment with the central position of the measured object, a deviation from actual temperature may be caused. It is suggested to repeat the measurement once or more at same positions

Imp.& Mkt. By:

REMEDY HEALTHCARE PRIVATE LIMITED

Regd. Add: S/A -1/20, Jogabai Extn., Behind - New Friends

Colony, OKHLA, New Delhi - 110025, [INDIA].

PH No :- +91- 11-65103600, Cell:- 8750059800.

Helpline No:- +91-9911959800 / **9911954800**

Email 1: **drcure.poct@gmail.com**

Email 2 : **remedyhealthcarepvtltd@gmail.com**

Technical Specifications:

Working Conditions	Environment temp: 16°C-35°C
	Relative temp: ≤85%
	Power: DC3V-2- AA Dry Cell
Display	Digital LCD
Unit	°C / °F
Resolution	0.1 Degree
Measurement range	Body temp:32.0°C-42.0°C
	Surface temp:0°C-110°C
Error	±0.3°C
Repeatability	±0.2°C
Timeconsuming	≤0.5second
Memory	32 Groups of memories
Measuring distance	50MM-100MM
Emission rate	0.95

