

REMEDY™
Good Health For All...

HAEMOCHECK™

Haemoglobin Color Scale

“CE” Registered Product



REMEDY HEALTH CARE PVT. LTD.

(An ISO 9001 : 2008 Certified Company)

Office: S/A-1/20, Jogabai Extn., B/h - New Friends Colony, Okhla, New Delhi - 110025.
Ph.: +91-11-65103600, Mob: 8750059800 / 8506059800 ; Help Line No: 9911954800.

E-mail: remedyhealthcarepvtltd@gmail.com / haemocheck@gmail.com

Web: www.remedyhealthcare.in

WHO REFERENCE

Haemoglobin equivalent g/dl	14	●
	12	●
	10	●
	8	●
	6	●
	4	●

14 g/dl	healthy
12 g/dl or more	not anemic
8-11 g/dl	mild to moderate anemia
6-7 g/dl	marked anemia
4-5 g/dl	severe anemia
Less than 4 g/dl	critical

HAEMOCHECK™ : Haemoglobin Color Scale



This Device Can be used in 3 - Simple Steps as under:

1. Prick the finger and obtain one free flowing drop of Blood on (smooth surface) of **HAEMOCHECK™** strip.
2. Wait for 30 seconds to 1 minute.
3. Match the color of Blood drop from lighter to darker shades of **HAEMOCHECK™** Color Scale.

- **HAEMOCHECK™** is the simplest device available for estimating the HB level of Anemic Patients in absence of Laboratory under Field Condition in Camps.
- **HAEMOCHECK™** kit contains “CE” marked Color Shade Card, Specially Imported test strips & Blood Lancets.
- The shades of the Color Scale Card represent the range of HB levels from 4 to 14 g/dl. Just by matching the color of a drop of blood [Fresh Capillary] on the special test strip provided, You can know the level of Haemoglobin in the blood.
- **HAEMOCHECK™** is best suited for field testing of Haemoglobin of children, pregnant women, Adolescent girl child and Adults with Malaria or other parasitic condition as well as for mass screening in CAMPS.
- Most accurate results than any other method for field testing as per various research & hence **Haemoglobin color scale Strip method is recommended by WHO for field testing.**
- Haemoglobin color scale is being used Internationally as well as by Govt. of India & many state Govt. agencies for Anemia Eradication Programs viz; NRHM, NGO, DHC, PHC, CHC, ASHA-Workers, Anganwadi-Workers, Medical Practitioners, Clinicians, Gynecologists & Blood Banks etc.
- **HAEMOCHECK™** is Marketed Worldwide by **REMEDY HEALTH CARE PVT. LTD.** An ISO 9001 : 2008 Certified Company.