

**REMEDY** <sup>TM</sup>  
*Good Health For All...*

# HAEMOTEST<sup>TM</sup>

Haemoglobin Color Scale

“CE” Registered Product



**REMEDY HEALTH CARE PVT. LTD.**

(An ISO 9001 : 2008 Certified Company)

**Corp. Office:** 48, 3rd Floor, Bharat Nagar, New Friends Colony, New Delhi - 110025 [INDIA].


**Regd. Office:** S/A-1/20, 1st Floor, Jogabai Extn., B/h - NFC, Okhla, New Delhi - 110025.

**Ph.:** +91-11-65103600, **Mob:** 8750059800 / 8506059800 ; **Help Line No:** 9911954800.

**E-mail:** [remedyhealthcarepvtltd@gmail.com](mailto:remedyhealthcarepvtltd@gmail.com) / [haemotest@gmail.com](mailto:haemotest@gmail.com)

**Web:** [www.remedyhealthcare.in](http://www.remedyhealthcare.in)

# WHO REFERENCE

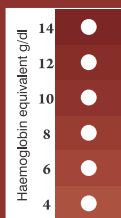

14 g/dl	healthy
12 g/dl or more	not anemic
8-11 g/dl	mild to moderate anemia
6-7 g/dl	marked anemia
4-5 g/dl	severe anemia
Less than 4 g/dl	critical

## HAEMOTEST™ : Haemoglobin Color Scale



This Device Can be used in 3 - Simple Steps as under:

1. Prick the finger and obtain one free flowing drop of Blood on **HAEMOTEST** strip.
2. Wait for 30 seconds to 1 minute.
3. Match the color of Blood drop from lighter to darker shades of **HAEMOTEST** Color Scale.

- **HAEMOTEST**™ is the simplest device available for estimating the HB level of Anemic Patients in absence of Laboratory under Field Condition in Camps.
- **HAEMOTEST**™ kit contains “CE” marked Color Shade Card, Specially Imported test strips & Blood Lancets.
- The shades of the Color Scale Card represent the range of HB levels from 4 to 14 g/dl. Just by matching the color of a drop of blood [Fresh Capillary] on the special test strip provided, You can know the level of Haemoglobin in the blood.
- **HAEMOTEST**™ is best suited for field testing of Haemoglobin of children, pregnant women, Adolescent girl child and Adults with Malaria or other parasitic condition as well as for mass screening in CAMPS.
- Most accurate results than any other method for field testing as per various research & hence Haemoglobin color scale Strip method is recommended by WHO for field testing.
- Haemoglobin color scale is being used Internationally as well as by Govt. of India & many state Govt. agencies for Anemia Eradication Programs viz; NRHM, NGO, DHC, PHC, CHC, ASHA-Workers, Anganwadi-Workers, Medical Practitioners, Clinicians, Gynecologists & Blood Banks etc.
- **HAEMOTEST**™ is Marketed Worldwide by **REMEDY HEALTH CARE PVT. LTD.**, An ISO 9001 : 2008 Certified Company.